


Absorbable Periodontal Dressing

Bright Oral-Pack

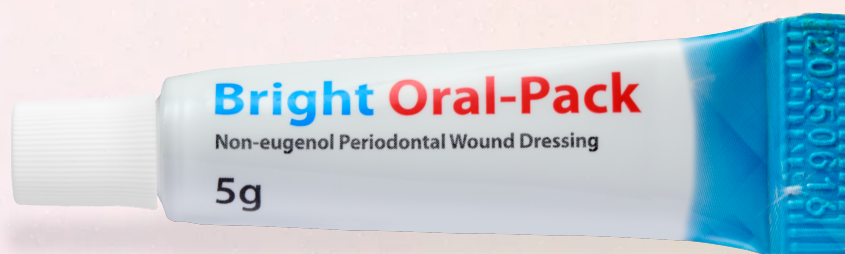

1 Ready-to-Use

No mixing required
Direct application from the tube



2 Excellent Retention

Reliable adhesion in moist surgical sites
Low expansion minimizes dislodgement



3 Esthetic Color Matching

Gingiva-like color
Natural post-op appearance



4 Naturally Resorbable

Maintains protection for 3-4 days
No removal required






Dentium

Indications

- Protection of surgical sites following flap surgery or Implant
- Post-operative management following soft tissue grafting and GBR procedures
- Protection of exposed extraction sites
- Barrier against external irritation at suture sites

Clinical Case

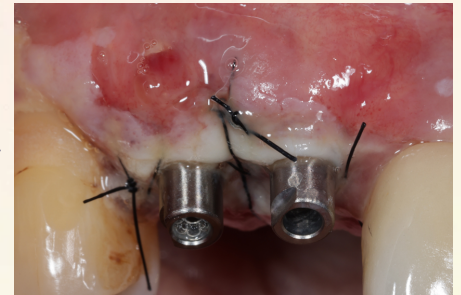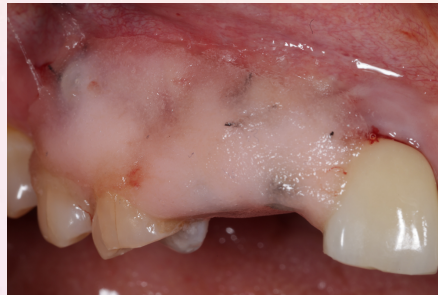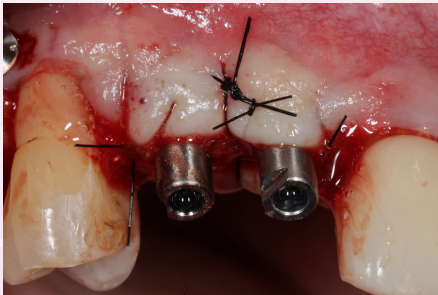

Implant placement and suturing

Application of **Bright Oral-Pack**



Healing status after 3 days

How to Use



- 1 Clean the surgical site with saline.
- 2 Moisten a gloved finger or instrument with sterile water or petroleum jelly, then apply directly.
- 3 Adapt gently over the sutured area with a wet gloved finger.
 - 💡 Using wet gauze may improve adaptation and handling.
- 4 Maintains for 3–4 days and gradually resorbs.

Order Information

Type	Model	Volume	Packaging
Tube Type	PDW005	5g	 5g x 5tubes
	PDW025	25g	 25g x 1tube